1.03981.0102 **REF**
1.03981.1000

Microscopy

M-FIX® spray fixative

for cytodiagnosis

For professional use only

In Vitro Diagnostic Medical Device

Intended purpose

This "M-FIX® spray fixative - for cytodiagnosis" is used for human-medical cell diagnosis and serves the purpose of the cytological fixation of sample material of human origin. It is a ready-to-use fixing solution that when used together with other *in vitro* diagnostic products from our portfolio makes target structures evaluable for diagnostic purposes (by fixing, followed by staining, counterstaining, mounting) in human gynecological and clinico-cytological specimen materials, for example cervical smears.

Using the auxiliary reagents from our portfolio creates the conditions that enable authorized and qualified investigators to make a correct diagnosis at the end of the diagnostic process. In this regard, auxiliary IVD reagents serve *inter alia* to process human specimen material (e.g. fixing, decalcifying, dehydrating, clarifying, paraffin-embedding, mounting, microscoping, archiving). When used together with the corresponding staining solutions, this enables the visualization of cellular structures that are otherwise low in contrast, thus rendering them evaluable under the optical microscope. Further examinations may be necessary to reach a definitive diagnosis.

Principle

A precondition for exact cytological diagnosis is the perfect fixation of sample material.

In order to prevent the cells from drying out and shrinking, to maintain the specimen's structural features and to enable a clear staining and differentiation of the specimen, the microscopic preparations must be fixed immediately after being taken and while still in a moist condition.

If specimens are fixed too late, the specimen contains single cells and cell clusters that show up as brownish, granular artefacts and may have a negative influence on the diagnostic work.

Sample material

Gynecological and non-gynecological specimens as sputum, urine, smears from fine needle aspiration biopsies (FNAB), effusions, rinses

Reagents

Cat. No. 1.03981.0102

M-FIX® spray fixative
for cytodiagnosis

100 ml

Cat. No. 1.03981.1000

M-FIX® spray fixative
for cytodiagnosis (refill pack)

1 l

Sample preparation

The sampling must be performed by qualified personnel.

All samples must be treated using state-of-the-art technology.

All samples must be clearly labeled.

Suitable instruments must be used for taking samples and their preparation. Follow the manufacturer's instructions for application / use.

Procedure

The still moist smear is sprayed immediately 3 times with M-FIX® spray fixative so that the specimen is evenly wetted, but not drenched.

Care should be taken that the spray is approximately 20 cm away from the object while spraying in order to avoid any potential loss of cells.

The alcohol contained in M-FIX® spray fixative evaporates, leaving polyethylene glycol on the specimen as a protective film. It is advisable to cover the label section of the slide with your finger during the spraying procedure, since the polyethylene glycol protective film may make it difficult to write on.

After approx. 10 min the specimen is dry and can now be stained, stored or dispatched for further processing. Specimens prepared in this manner remain stable for a period of several weeks.

During staining, care should be taken that the specimen is first immersed in distilled water or 50 % ethanol for approx. 10 sec for removing the polyethylene glycol film.

The descending alcohol series before the hematoxylin step in the Papanicolaou method can be omitted.

Technical notes

The microscope used should meet the requirements of a medical diagnostic laboratory.

When using automatic staining systems, please follow the instructions for use supplied by the supplier of the system and software.

Remove surplus immersion oil before filing.

Analytical performance characteristics

The present auxiliary reagent "M-FIX® spray fixative" aids in the microscopic examination of biological structures as described in the "Intended purpose" of this IFU. The use of the product is only to be carried out by authorized and qualified persons, this includes, among other things, sample and reagent preparation, sample handling, decisions regarding suitable controls and more.

The analytical performance of the product is confirmed by testing each production batch.

For the following stains, the analytical performance was confirmed in terms of specificity, sensitivity and repeatability of the product with a rate of 100 %:

	Inter-assay Specificity	Inter-assay Sensitivity	Intra-assay Specificity	Intra-assay Sensitivity
Cytological staining				
Suitability for fixative	20/20	20/20	6/6	6/6

Analytical performance results

Intra- (performed on the same batch) and inter-assay (performed on different batches) data list the number of correctly stained structures in relation to the number of performed assays.

The results of this Performance Evaluation confirms that the product is suitable for the intended use and performs reliably.

Diagnostics

Diagnoses are to be made only by authorized and qualified personnel. Valid nomenclatures must be used.

This product is an auxiliary reagent that, when used together with other IVD products such as staining solutions, renders human specimen material evaluable for diagnostic purposes. Further tests must be selected and implemented according to recognized methods.

Suitable controls should be conducted with each application in order to avoid an incorrect result.

Storage

Store the M-FIX® spray fixative - for cytodiagnosis at +15 °C to +25 °C.

Any crystallization that may occur during transport or due to storage at varying temperatures can be easily resolved by shaking the bottle vigorously.

Shelf-life

The M-FIX® spray fixative - for cytodiagnosis can be used until the stated expiry date.

After first opening of the bottle, the contents can be used up to the stated expiry date when stored at +15 °C to +25 °C.

The bottles must be kept tightly closed at all times.

Capacity

100 ml M-FIX® spray fixative is sufficient for approx. 1000 - 1250 applications.

Additional instructions

For professional use only.

In order to avoid errors, the application must be carried out by qualified personnel only.

National guidelines for work safety and quality assurance must be followed. Microscopes equipped according to the standard must be used.

If necessary use a standard centrifuge suitable for medical diagnostic laboratory.

Protection against infection

Effective measures must be taken to protect against infection in line with laboratory guidelines.

Instructions for disposal

The package must be disposed of in accordance with the current disposal guidelines.

Used solutions and solutions that are past their shelf-life must be disposed of as special waste in accordance with local guidelines. Information on disposal can be obtained under the Quick Link "Hints for Disposal of Microscopy Products" at www.microscopy-products.com. Within the EU the currently applicable REGULATION (EC) No 1272/2008 on classification, labelling and packaging of substances and mixtures, amending and repealing Directives 67/548/EEC and 1999/45/EC, and amending Regulation (EC) No 1907/2006 applies.

Auxiliary reagents

Cat. No. 1.00974	Ethanol denatured with about 1 % methyl ethyl ketone for analysis EMSURE®	1 l, 2.5 l
Cat. No. 1.04699	Immersion oil for microscopy	100-ml dropping bottle, 100 ml, 500 ml
Cat. No. 1.05175	Hematoxylin solution modified acc. to Gill II for microscopy	500 ml, 2.5 l
Cat. No. 1.06887	Papanicolaou's solution 2b Orange II solution for cytology	500 ml, 2.5 l
Cat. No. 1.06888	Papanicolaou's solution 2a Orange G solution (OG 6) for cytology	500 ml, 1 l, 2.5 l
Cat. No. 1.07961	Entellan™ new rapid mounting medium for microscopy	100 ml, 500 ml, 1 l
Cat. No. 1.08298	Xylene (isomeric mixture) for histology	4 l
Cat. No. 1.09016	Neo-Mount™ anhydrous mounting medium for microscopy	100-ml dropping bottle, 500 ml
Cat. No. 1.09253	Papanicolaou's solution 1a Harris hematoxylin solution for cytology	500 ml, 1 l, 2.5 l
Cat. No. 1.09254	Papanicolaou's solution 1b Hematoxylin solution S for cytology	500 ml, 2.5 l
Cat. No. 1.09269	Papanicolaou's solution 3d polychromatic solution EA 65 for cytology	100 ml, 2.5 l
Cat. No. 1.09270	Papanicolaou's solution 3c polychromatic solution EA 65 for cytology	100 ml
Cat. No. 1.09271	Papanicolaou's solution 3a polychromatic solution EA 31 for cytology	500 ml, 2.5 l
Cat. No. 1.09272	Papanicolaou's solution 3b polychromatic solution EA 50 for cytology	500 ml, 1 l, 2.5 l
Cat. No. 1.09843	Neo-Clear™ (xylene substitute) for microscopy	5 l

H225: Highly flammable liquid and vapor.
H319: Causes serious eye irritation.

P210: Keep away from heat, hot surfaces, sparks, open flames and other ignition sources. No smoking.

P233: Keep container tightly closed.

P240: Ground and bond container and receiving equipment.

P241: Use explosion-proof electrical/ ventilating/ lighting/ equipment.

P242: Use non-sparking tools.

P305 + P351 + P338: IF IN EYES: Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do. Continue rinsing.

Revision History

Version	Modification Comment
2024-Aug-01	Initial version with the introduction of Revision History

Hazard classification

Cat. No. 1.03981

Please observe the hazard classification printed on the label and the information given in the safety data sheet.

The safety data sheet is available on the website and on request.

Main components of the product

Cat. No. 1.03981

C_2H_5OH 875 g/l
 $HO(C_2H_4O)_nH$ 50 g/l

General remark

If during the use of this device or as a result of its use, a serious incident has occurred, please report it to the manufacturer and / or its authorised representative and to your national authority.

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Catalog number

Batch code

Caution, consult accompanying documents

Use by YYYY-MM-DD

Temperature limitation

Status: 2024-Aug-01 V.2

MilliporeSigma is the U.S. and Canada Life Science business of Merck KGaA, Darmstadt, Germany.

© 2024 Merck KGaA, Darmstadt, Germany and/or its affiliates. All Rights Reserved. MilliporeSigma and Sigma-Aldrich are trademarks of Merck KGaA, Darmstadt, Germany. All other trademarks are the property of their respective owners. Detailed information on trademarks is available via publicly available resources.

EMD Millipore Corporation, 400 Summit Drive, Burlington MA 01803, USA, Tel. +1-978-715-4321
MilliporeSigma Canada Ltd., 2149 Winston Park Dr, Oakville, Ontario, L6H 6J8, Canada, Phone: +1 800-565-1400
www.sigmaaldrich.com

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- Histological and Histochemical Methods, Theory and practice, J.A. Kiernan, 2015, Scion Publishing Ltd, 5th Edition